

Operator's Manual for: *NL* **Millennium 2**

Public Use Wheelchair Lifts

DOT — Public Use Lift

"DOT — Public Use Lift" verifies that this platform lift meets the "public use lift" requirements of FMVSS No. 403. This lift may be installed on all vehicles appropriate for the size and weight of the lift, but must be installed on buses, school buses, and multi-purpose passenger vehicles other than motor homes with a gross vehicle weight rating (GVWR) that exceeds 4,536 kg (10,000 lb).



"Providing Access to the World"®

International Corporate Hdqrs: P.O. Box 310 Winamac, IN 46796 USA
1-800-THE LIFT® (574) 946-6153 FAX: (574) 946-4670



⚠ WARNING



Read manual before operating lift. Failure to do so may result in serious bodily injury and/or property damage. Keep manual in lift storage pouch.

Congratulations

We at The Braun Corporation wish to express our fullest appreciation on your new purchase. With you in mind, our skilled craftsmen have designed and assembled the finest lift available.

This manual includes safety precautions, lift operating instructions, manual operating instructions, and instructions for maintenance and lubrication procedures.

Your lift is built for dependability, and will bring you years of pleasure and independence, as long as maintenance is performed regularly and the lift is operated by an instructed person.

Sincerely,

THE BRAUN CORPORATION

A handwritten signature in black ink, reading "Ralph W. Braun". The signature is written in a cursive style with a large, sweeping "B" and a trailing flourish.

Ralph W. Braun
Chief Executive Officer

Contents

Warranty/Registration Instructions	2, 3	Outer Barrier Latch	20, 21
Lift Terminology		Outer Barrier and Latch Operation	21
Lift Terminology Illustration	4	Bridging	21
Introduction	5	Handrails	21
Direction	5	Lift Passengers	
Lift Components	6, 7	Passenger Orientation (Boarding Direction)	22
Vehicle and Lift Interlocks	7	Standeers	22
Lift Actions and Functions	8	Yellow Boundaries	22, 23
Lift Operation Safety		Vehicle (Floor Level) Loading and	
Safety Symbols	9	Unloading	23, 24
Lift Operation Safety Precautions	10-13	Wheelchair-Equipped Occupant Seat Belts	25
Operation Notes and Details		Operation Procedure Review	25, 26
Introduction/Lift Access Doors and Interlocks	14	Preventive Maintenance	26
General Safety	15	Lift Operating Instructions	27-31
Control Switches	15	NHTSA Operations Checklist	32
Lift Features		Manual Operating Instructions	33-35
Lift-Tite™ Latches	16, 17	Decals and Antiskid	36-38
Inner Roll Stop	18, 19	Maintenance and Lubrication	39-47
Outer Barrier	19, 20	Limited Warranty	48-55

Warranty and Registration Instructions

Immediately upon receiving your lift, examine the unit for any damage. Notify the carrier at once with any claims.

Two warranty/registration cards are located in the lift-mounted owner's manual storage pouch (shown on opposite page). The sales representative must process one of the cards. The consumer

must fill out the other card and mail it to The Braun Corporation. The limited warranty is provided in this manual. The warranty cards must be processed to activate the warranty.

Two Braun Serial No./Series No. identification tags are posted on the lift (shown on opposite page). One I.D. tag is posted on the op-

posite pump side vertical arm. A second I.D. tag is located on the opposite pump side tower. Both I.D. tags provide the product identification information appearing on the Warranty/Registration card. Record the identification information in the space provided. This information must be provided when filing a warranty claim or ordering parts.

Model No. Serial No.

Sample Warranty/Registration Card

Record identification information below.

Model No. _____

Serial No. _____

Date of Manufacture _____

Sample Serial
No./Series No.
Identification Tag


The Braun Corporation
1-800-THE-LIFT™
BRAUNLIFT.COM™

DOT Public Use Lift MODEL#
XXXXXXXXXX

SERIAL NUMBER
XX-XXXXX

MFG DATE
XX/XX/XXXX

CE

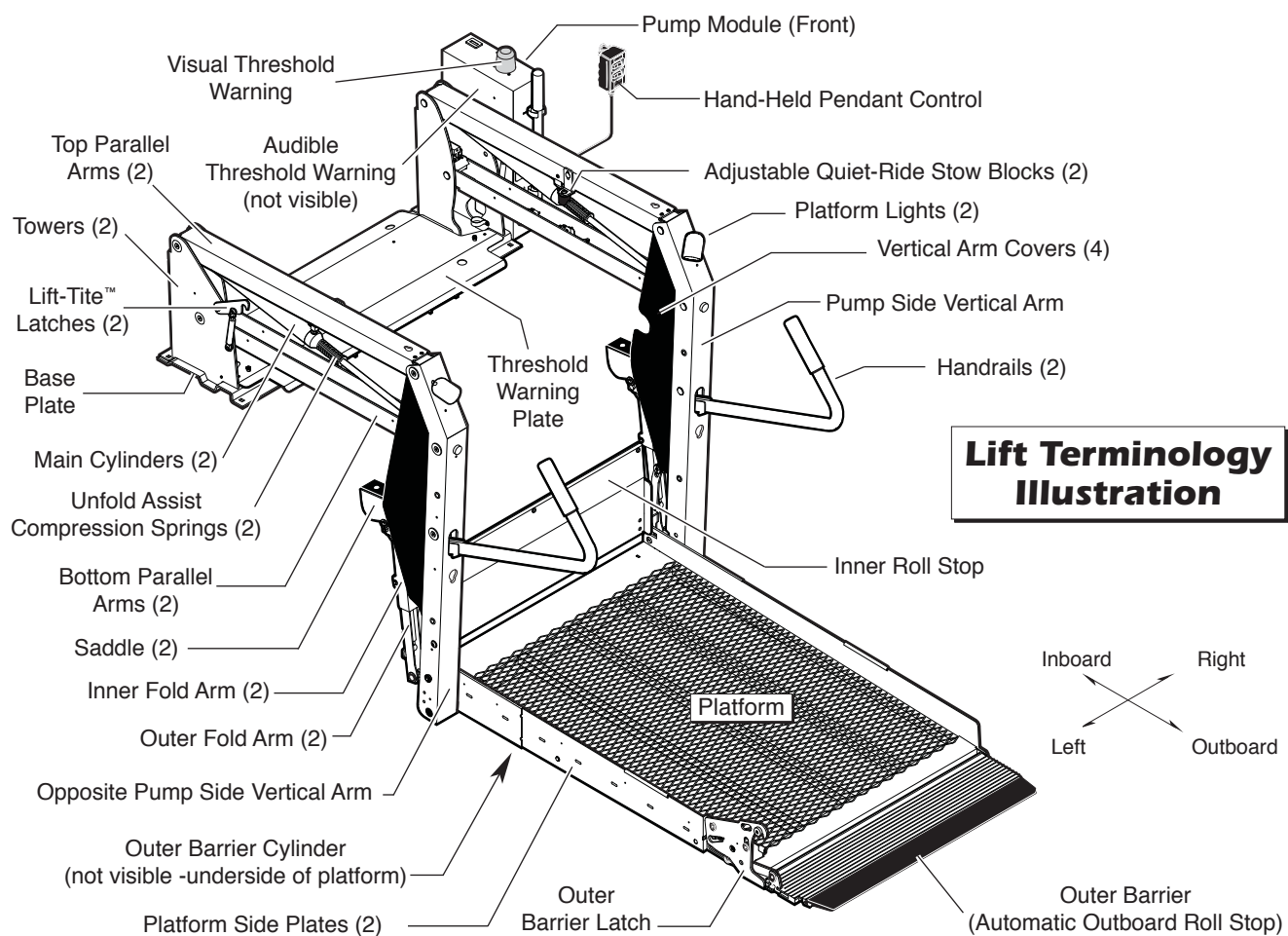

Lift Terminology

Introduction:

Braun NL-2 Millennium Series lifts are ADA compliant and comply fully with National Highway Traffic Safety Administration (NHTSA) specifications. The NL-2 is commercial oriented (intended for operation by an attendant).

Lift models vary per size (lift model numbers indicate lift dimensions). NL-2 Series lift models can be equipped with left or right side pump modules as needed. A right side pump-equipped lift model is depicted in the Lift Terminology Illustration (NL917FIB-2). Left side pump lift models are a mirrored image of front pump models

(pump module located on opposite end of base plate).

Refer to the Lift Terminology Illustration for identification of lift components.

Lift operation procedures are identical for all NL-2 Series lift models. The operating instructions contained in this manual and appearing on lift-posted operating instructions decals address the lift control switches and the corresponding lift functions. Instructions are provided for manual operation of the lift in event of power or equipment failure.

Terminology: Become familiar with the terminology that will be used throughout this manual. Become familiar with the identification of lift components and their functions. Contact your lift sales representative or call The Braun Corporation at 1-800-THE LIFT® if any of this information is not fully understood.

Direction: The terms "left," "right," "inboard," and "outboard" will be used throughout this manual to indicate direction (as viewed from outside the vehicle looking directly at the lift). Refer to the Lift Terminology Illustration for clarification of direction terms.

Lift Terminology

Lift Components

Refer to the Lift Terminology Illustration on page 4.

Pump Module: The lift-mounted pump module consists of the hydraulic pump, the manual hand pump, and electrical components that power the lift electric/hydraulic systems.

Hand-held Pendant Control: The hand-held attendant's pendant control is connected to the pump module. The hand-held pendant is equipped with two rocker switches, (UP/DOWN and FOLD/UNFOLD). The momentary switches activate the automatic lift functions.

Lift Frame: The lift frame consists of the base plate, threshold warning plate, towers, parallel arms, vertical arms, platform pivot arms and handrails. Two main hydraulic cylinders are housed in the parallel arms. The electrical/hydraulic powered lift frame components mechanically unfold, lower, raise and fold the lift platform assembly.

Platform Assembly: The platform assembly consists of the steel tubing frame with grating surface upon which the wheelchair is positioned, the outer barrier, outer barrier latch, the inner roll stop, and the hydraulic cylinder assembly that powers the outer barrier.

Lift-Tite™ Latches: The spring-loaded latches prevent the platform from unfolding from the stowed position in the event of platform drift. Further details regarding Lift-Tite™ latches are provided on pages 16 and 17.

Outer Barrier: The cylinder-powered automatic outer barrier provides a ramp for wheelchair loading and unloading at ground level. Photos and further details regarding the outer barrier are provided in the Operation Notes section (pages 19-21).

Outer Barrier Latch: The spring-loaded latch locks the outer barrier in the vertical position when the platform raises above ground level.

Lift Terminology

Inner Roll Stop: NL-2 Series lift models are equipped with an automatic inboard roll stop that also serves as the bridge plate. The roll stop bridges the gap between the lift platform and the vehicle floor. The inner roll stop automatically rotates from the horizontal position to the vertical position as the lift lowers and raises. Further details regarding the automatic mechanical inboard roll stop are provided on pages 18 and 19.

Vehicle and Lift Interlocks

Braun Corporation NL917IB-2 Series lifts comply fully with all NHTSA vehicle and lift interlock specifications. Vehicle movement is prohibited unless the lift is fully stowed and the lift will not function unless the vehicle is parked and secured.

The NL-2 features a visual and audible threshold warning system that will activate if the threshold area is occupied when the platform is one inch or more below floor level.

The inner roll stop and outer barrier sense weight to prohibit lift operation. The lift will not function if the inner roll stop or

the outer barrier are occupied. The lift platform cannot be folded (stowed) if occupied.

The inner roll stop features a locking mechanism that prohibits the platform from lowering if the lock does not engage. The lift platform cannot be raised or lowered unless the outer barrier is deployed (up).

Lift Terminology

Lift Actions and Functions

DEPLOY (A-C)

A. UNFOLD (Out) - Platform

Unfold: Unfold is the action of the platform rotating out and down from the fully-stowed (vertical) position to floor level (horizontal) position when the UNFOLD switch is pressed.

B. DOWN - Platform Lower:

Down is the action of the platform lowering from floor level position to fully-lowered (ground level) position when the DOWN switch is pressed.

C. DOWN - Outer Barrier Unfold (Down) - When the platform reaches the fully-lowered

(ground) position and the DOWN switch is continued to be pressed, the outer barrier rotates downward from vertical position to ramp position.

STOW (D-F)

D. UP - Outer Barrier Fold

(Raise): When the lift is fully lowered and the outer barrier is in the ramp position, pressing the UP switch first rotates the outer barrier upward from ramp position to vertical position.

E. UP - Platform Raise: Up is the action of the platform raising from ground level to floor level (fully-raised) position when the UP switch is pressed.

F. FOLD (In) - Platform Fold:

Fold is the action of the platform rotating up and in from the floor level (horizontal) position to fully-stowed (vertical) position when the FOLD switch is pressed.

Stowed Position: The lift is stowed when the lift platform has been fully raised and folded fully (vertical position).

Floor Level: Floor level is the position (height) the platform assembly reaches in order for the wheelchair passenger to enter and exit the vehicle (fully raised). The platform automatically stops at floor level when unfolding from the stowed position and when raising from ground level.

Lift Operation Safety

Safety Symbols

SAFETY FIRST! Know That....

A All information contained in this manual and supplements (if included), is provided for your safety. Familiarity with proper operation instructions as well as proper maintenance procedures are necessary to ensure safe, trouble free operation. Safety precautions are provided to identify potentially hazardous situations and provide instruction on how to avoid them.

B

WARNING

This symbol indicates important safety information regarding a potentially hazardous situation that could result in serious bodily injury and/or property damage.

C

CAUTION

This symbol indicates important information regarding how to avoid a hazardous situation that could result in minor personal injury or property damage.

D

Note: Additional information provided to help clarify or detail a specific subject.

These symbols will appear throughout this manual as well as on the labels posted on your lift. **Recognize the seriousness of this information.**

Lift Operation Safety

Lift Operation Safety Precautions

WARNING

If the lift operating instructions, manual operating instructions and/or lift operation safety precautions are not fully understood, contact The Braun Corporation immediately. Failure to do so may result in serious bodily injury and/or property damage.

 **WARNING** Read manual and supplement(s) before operating lift. Read and become familiar with all safety precautions, operation notes and details, operating instructions and manual operating instructions before operating the lift. Note: Wheelchair passengers and all transit agency personnel (drivers and wheelchair lift attendants) must read and become familiar with the contents of this manual and supplement(s) before operation.

 **WARNING** Load and unload on level surface only.

 **WARNING** Engage vehicle parking brake before operating lift.

 **WARNING** Provide adequate clearance outside the vehicle to accommodate the lift before opening lift door(s) or operating lift.

 **WARNING** Inspect lift before operation. Do not operate lift if you suspect lift damage, wear or any abnormal condition.

 **WARNING** Keep operator and bystanders clear of area in which the lift operates.

Lift Operation Safety

WARNING

Whenever a wheelchair passenger (or standee) is on the platform, the:

- Passenger must be positioned fully inside yellow boundaries
- Wheelchair brakes must be locked
- Inner roll stop and outer barrier must be up (vertical)
- Outer barrier latch must be fully engaged
- Passenger should grip both handrails (if able).

WARNING

Load and unload clear of vehicular traffic.

WARNING

Do not overload or abuse. The load rating applies to both the raising and lowering functions - continuous lifting capacity is 800 lbs.

WARNING

Discontinue lift use immediately if any lift or vehicle interlock does not operate properly.

WARNING

Do not operate or board the lift if you or your lift operator are intoxicated.

WARNING

Do not raise front wheelchair wheels (pull wheelie) when loading (boarding) the platform.

WARNING

Open lift door(s) fully and secure before operating lift.

WARNING

Position and secure (buckle, engage, fasten, etc.) the wheelchair-equipped occupant seat belt (torso restraint) before loading onto the wheelchair lift platform.

Lift Operation Safety

Lift Operation Safety Precautions (continued)

WARNING

If the lift operating instructions, manual operating instructions and/or lift operation safety precautions are not fully understood, contact The Braun Corporation immediately. Failure to do so may result in serious bodily injury and/or property damage.

-  **WARNING** Lift attendants must ensure that lift occupants keep hands, arms and all other body parts within the lift occupant area and clear of moving parts.
-  **WARNING** Platform must be positioned at floor level (bridging position) when loading or unloading in and out of vehicle.
-  **WARNING** Do not use platform inner roll stop or outer barrier as a brake. Stop and brake wheelchair when loading onto the platform (manually stop and brake manual wheelchairs — stop powered wheelchairs with the wheelchair controls).
-  **WARNING** Turn powered (electric) wheelchairs off when on lift platform.
-  **WARNING** Press the DOWN switch until the entire platform rests on ground level (lowered fully) and the outer barrier is fully unfolded (ramp position) before loading or unloading a passenger at ground level.

-  **WARNING** Outer barrier must be fully unfolded (ramp position) until front and rear wheelchair wheels cross the barrier when loading or unloading at ground level.

Lift Operation Safety

- ⚠ WARNING** Accidental activation of control switch(es) may cause unintended operation(s).
- ⚠ WARNING** Maintenance and lubrication procedures must be performed as specified in this manual by authorized (certified) service personnel.
- ⚠ WARNING** Replace missing, worn or illegible decals.
- ⚠ WARNING** Keep owner's (operator's) manual in lift-mounted manual storage pouch at all times.
- ⚠ WARNING** Never modify (alter) a Braun Corporation lift.
- ⚠ WARNING** Do not use accessory devices not authorized by The Braun Corporation.
- ⚠ WARNING** Do not remove any guards or covers.
- ⚠ WARNING** Keep clear of any hydraulic leak.
- ⚠ WARNING** Failure to follow these safety precautions may result in serious bodily injury and/or property damage.

Operation Notes and Details

WARNING

Read and become familiar with all lift operation safety precautions, operation notes and details, operating instructions and manual operating instructions prior to operating the lift. If this information is not fully understood, contact The Braun Corporation immediately. Failure to do so may result in serious bodily injury and/or property damage.

NL-2 Series “Public Use” lift models are specifically designed to be operated by an attendant. The Lift Operating Instructions contained in this manual and posted on the lift provide instructions for operation of the lift only. Read and become familiar with all lift operation safety precautions, operation notes and details, operating instructions and manual operating instructions before attempting lift operation procedures.

Lift Access Doors and Interlocks: Attendants must become familiar with the vehicle lift access door system and vehicle interlock system(s). Transit vehicles and lift access door configurations vary. Door securement devices (latches, hooks, cables, etc.) and procedures to operate them vary.

Instructions for operation of vehicle interlocks and door securement systems are not addressed in this manual or on lift-posted operating instructions decals due to the variety of procedures required for operating them. Braun Corporation NL-2 Series lifts comply fully with all NHTSA vehicle and lift interlock specifications (detailed on pages 7 and 32). Infringement of any lift interlock will prohibit lift operation.

It is the responsibility of the lift operator (attendant) to properly open, secure and close the vehicle lift door(s), to activate the vehicle interlock(s), to load and unload the wheelchair passenger (or standee) on and off the lift platform, and to properly activate all lift functions.

Operation Notes and Details

General Safety: The lift operator (attendant) and bystanders must keep clear of the area in which the lift operates and clear of all moving parts. Lift attendants must ensure that lift occupants (passengers) keep hands, arms and all other body parts within the lift occupant area and clear of moving parts.

Control Switches




Hand-held Pendant

Lift Power ON/OFF Switch: This switch must be in the ON position in order to activate the lift. The green Power Indicator Light mounted on the outboard face of the pump module illuminates to signal power to the lift.

Hand-held Pendant Control: The hand-held attendant's pendant control is equipped with two rocker switches, (UP/DOWN and FOLD/UNFOLD). The momentary switches activate the automatic lift functions. Simply press the switch labeled for the intended function. When there is power to the lift, the lift function labels illuminate to identify the functions.

Lift Power Switch On/Off Switch

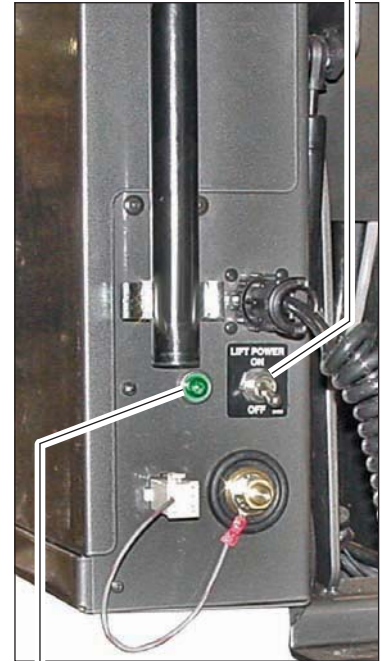

Power Indicator Light

Operation Notes and Details

Lift Features

Become familiar with all lift features and the proper operation of the lift components before attempting lift operation. Refer to the Lift Terminology Illustrations


for identification of specific lift components if not clearly depicted in this section. Contact The Braun Corporation at 1-800-THE LIFT® immediately if any of this

information is not understood.

Lift-Tite™ Latches: NL-2 Series lifts are equipped with Lift-Tite™ Latches. Lift-Tite™ Latches pre-

WARNING

Discontinue lift use immediately if any lift component does not operate properly. Failure to do so may result in serious bodily injury and/or property damage.

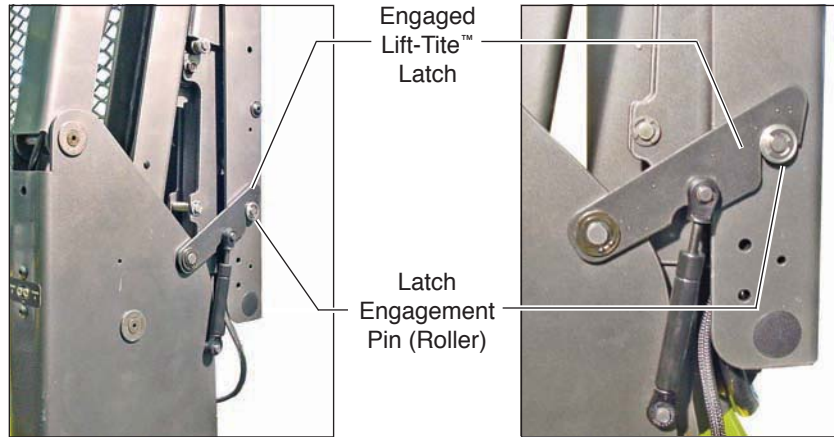

Operation Notes and Details


vent the platform from unfolding from the stowed position in the event of platform drift. Due to the “all-hydraulic” operation of the dual-cylinder NL-2, hydraulic fluid expansion, contraction or seepage may occur. Any of these conditions may result in platform drift (failure to hold the platform in the folded or raised position). Platform drift may occur during lift shipment and/or between extended periods of non-lift use.

In the event that the platform does not unfold when the UNFOLD switch is pressed, press the FOLD switch momentarily to disengage the Lift-Tite™ latches (platform drift has occurred). Then, press the UNFOLD switch to unfold the

platform to floor level (standard operation).

When manually operating a lift equipped with Lift-Tite™ latches, insert the pump handle in the pump and stroke until the platform folds fully (stops). Then,

open the hand pump valve (turn counterclockwise) to unfold the platform. Folding the platform fully first will ensure that the Lift-Tite™ latches will disengage properly when the release valve is opened. Manual Operating Instructions are provided on pages 33-35.

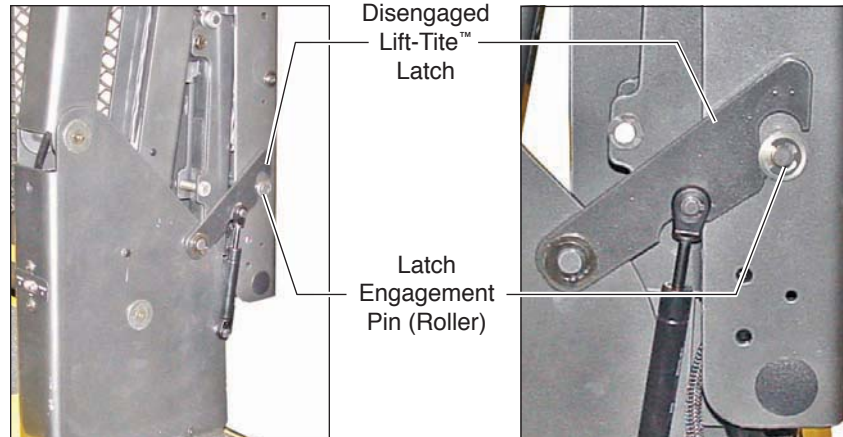

Operation Notes and Details

Lift Features (continued)

Inner Roll Stop: NL-2 Series lift models are equipped with an automatic inner roll stop that also serves as the bridge plate (inner roll stop photos below).

When the UNFOLD switch is

pressed and the platform unfolds from stow position to floor level, this mechanical roll stop is automatically unfolded to the bridging (horizontal) position to provide a bridge plate between the platform and the lift base plate (vehicle floor). The inboard edge of the bridge plate rests on the threshold warning plate (base plate).

The roll stop automatically deploys (rotates) to the vertical (roll stop) position as the platform lowers to the ground (DOWN switch is pressed).

As the UP switch is pressed and the platform raises from ground level, the inner roll stop automatically unfolds (rotates) to the



Fully-Deployed (vertical) Inner Roll Stop

Fully-Unfolded (horizontal) Inner Roll Stop (bridging position)



The inner roll stop must overlap the threshold (base) plate a minimum 1/2".



Operation Notes and Details

WARNING

Discontinue lift use immediately if any lift component does not operate properly. Failure to do so may result in serious bodily injury and/or property damage.

horizontal (bridging) position when it reaches vehicle floor level. The inner roll stop must overlap the threshold (base) plate a minimum 1/2".

When the FOLD switch is pressed and the platform folds from floor level to the stow

(vertical) position, the inner roll stop automatically travels inboard to the stowed position.

Interlock Features: The inner roll stop features a locking mechanism that prohibits the platform from lowering if the lock is not engaged. The inner roll stop also senses weight to prohibit lift operation. The lift will not function if inner roll stop is occupied.

Discontinue lift operation immediately if the inner roll stop does not operate properly.

Outer Barrier: This cylinder-powered barrier provides a ramp for wheelchair loading and unloading at ground level (see photos on following page). When the platform

lowers fully to ground level, the cylinder compression spring automatically unfolds (rotates) the barrier to the ramp position (fully unfolded).

Although the outer barrier is lift-powered, the activation of the barrier is controlled by the lift operator (attendant). Pressing the DOWN switch unfolds the barrier. The outer barrier is cylinder-powered to automatically fold (rotate) to the vertical position as shown when the UP switch is pressed (barrier raises before platform raises).

Interlock: The outer barrier senses weight to prohibit lift operation. The lift will not function if the outer barrier is occupied.

Operation Notes and Details

Lift Features (continued)


Discontinue lift operation immediately if the outer barrier does not operate properly.

Outer Barrier Latch: A spring-

loaded latch engages the outer barrier when the barrier rotates upward from ramp position to the vertical position (UP switch is pressed to raise the platform above ground level). Note: The platform must raise approximately three inches before the latch

engages fully (see photos on previous page).

Interlock: Lift platform movement shall be interrupted unless the outer barrier is raised (vertical position) and the outer barrier latch is positively engaged.

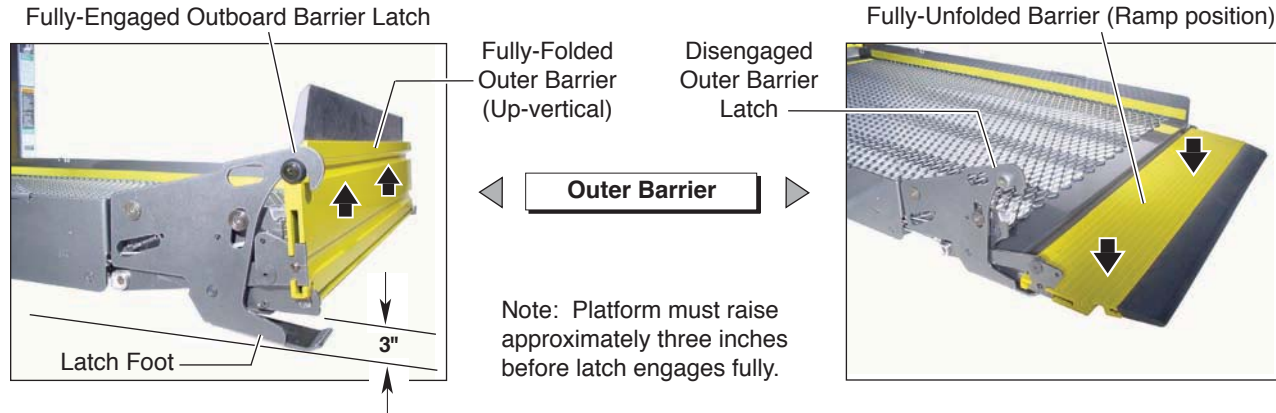

Operation Notes and Details

The latch disengages the barrier when the platform lowers fully (reaches ground level) and the latch foot contacts the ground (latch raises above the barrier).

Outer Barrier and Latch Operation: The lift attendant must press the DOWN switch to lower the platform fully to the ground. The attendant must view the platform as it lowers to be certain the entire platform reaches and rests safely on the ground. Stop pressing the DOWN switch if any portion of the platform is obstructed while descending or the entire platform does not reach ground level for any reason (contact with an obstruction, mechanical failure, exceeding the lift “floor-to-ground” capacity, etc.).

After the platform is fully lowered, the attendant should continue to press the DOWN switch to unfold the outer barrier fully (ramp position). Note: The barrier must be fully unfolded until the entire wheelchair (or standee) has crossed the barrier when loading or unloading at ground level. Discontinue lift operation immediately if the outer barrier or latch do not operate properly.

Bridging: The NL-2 incorporates a bridging feature. This feature stops the down travel of the platform if the outboard end of the platform contacts a raised surface (such as a curb), preventing the operator from lowering the inboard end of the platform.

Handrails: Dual handrails are provided for wheelchair passenger (or standee) use. The handrails unfold automatically to the horizontal position when the lift unfolds and automatically fold to the vertical position when the platform folds.

If able, passengers should grip both handrails when on the lift platform. Discontinue lift operation immediately if the handrails do not operate properly.

Operation Notes and Details

Lift Passengers

If you are an attendant operating the lift, it is your responsibility to perform safe loading and unloading procedures. Wheelchair lift attendants should be instructed on any special needs and/or procedures required for safe transport of wheelchair passengers.

The lift operator and bystanders must keep clear of the area in which the lift operates. Observe your passenger at all times during lift operation.

Do not attempt to load or unload a passenger in a wheelchair or other apparatus that does not fit on the platform area. Do not exceed the 800 pound load capacity

of the lift. The lift attendant should not ride on the platform with the passenger.

Passenger Orientation (Boarding Direction): Braun NL-2 Series wheelchair lifts accommodate both inboard and outboard facing wheelchair passengers or standees. Inboard facing of wheelchair lift passengers is not prohibited, but outboard facing of passengers is recommended by The Braun Corporation.

Braun NL-2 Series lifts permit both inboard and outboard facing of wheelchair and mobility aid users and accommodate persons using walkers, crutches, canes or braces or who otherwise have difficulty using steps.

Standees: Lift Operating Instructions apply to wheelchair passengers and standees. Standees should stand in the center of the platform (fully inside the yellow boundaries) and grip both handrails (if able) when on platform.

Yellow Boundaries: The passenger must be positioned in the center of the platform to prevent side-to-side load imbalance. The lift attendant (operator) should not ride on the platform with the passenger.

Yellow platform loading boundaries are identified in the following manner. The yellow powder-coated inner roll stop and outer barrier identify the inboard and outboard ends of the platform. A yellow powder-coated guard is

Operation Notes and Details

located on the pump-side platform side plate. A yellow boundary strip decal is affixed to the opposite platform side plate. Yellow plastic caps are placed on the dual handrails.

The threshold warning plate is powder-coated yellow to identify the platform threshold area.

Interlock: A visible and audible threshold warning system will activate if the threshold area is occupied when the platform is one inch or more below floor level.

The attendant must always be certain the wheelchair passenger or standee is properly positioned on the platform (fully inside yellow boundaries) and the wheelchair brakes are locked when a

passenger is on the lift platform. The lift passenger must keep hands, arms and all other body parts within the lift occupant area and clear of all moving parts.

Vehicle (Floor Level) Loading and Unloading: The platform must be fully raised (at floor level) and the inner roll stop (bridge plate) must be properly positioned when loading or unloading passengers in or out of the vehicle. It is the responsibility of the lift attendant to ensure the platform and the inner roll stop are properly positioned at floor level when loading and unloading passengers.

WARNING

Whenever a passenger is on the platform, the:

- Passenger must be positioned fully inside yellow boundaries
- Wheelchair brakes must be locked
- Inner roll stop and outer barrier must be UP

Failure to follow these rules may result in serious bodily injury and/or property damage.

Operation Notes and Details

Lift Passengers (continued)

The wheelchair brakes must be locked, the outer barrier must be in the fully-up (vertical) position and the outer barrier latch must be fully engaged whenever a passenger is on the platform.

Do not use the outer barrier as a brake. Stop and brake the wheelchair when fully loaded on the platform. Manually stop and brake manual wheelchairs. Stop powered wheelchairs with the wheelchair controls. Turn powered (electric) wheelchairs off when on the platform.

If the outer barrier, inner roll stop, handrails or any other lift component does not operate as outlined in this manual, discontinue lift use immediately and contact The Braun Corporation sales representative in your area or call The Braun Corporation at 1-800-THE LIFT®. One of our national Product Support representatives will direct you to an authorized service technician who will inspect your lift.

WARNING

Discontinue lift use immediately if any lift component does not operate properly. Failure to do so may result in serious bodily injury and/or property damage.

Operation Notes and Details

Wheelchair-Equipped Occupant Seat Belts

The Braun Corporation recommends wheelchair passengers position and buckle their wheel-

WARNING

Position and secure (buckle, engage, fasten, etc.) the wheelchair-equipped occupant seat belt before loading onto the wheelchair lift platform. Failure to do so may result in serious bodily injury and/or property damage.

chair-equipped seat belt (torso restraint) before loading onto a wheelchair lift.

Different types of disabilities require different types of wheelchairs and different types of wheelchair-equipped occupant restraint belt systems (torso restraints). It is the responsibility of the wheelchair passenger to have his or her wheelchair equipped with an occupant restraint (seat belt) under the direction of their health care professional.

Wheelchair lift attendants should be instructed on any special needs and/or procedures required for safe transport of wheelchair passengers.

Operation Procedure Review

The Braun Corporation recommends that transit agency supervisors and wheelchair lift attendants review the safety precautions and operation procedures appearing in this manual and on lift-posted decals with your wheelchair lift sales representative (dealer), before attempting lift operation.

Any questions or concerns can be addressed by the sales representative at that time. Operate the lift through all functions with your sales representative on hand to ensure the proper use and operation of the wheelchair lift is understood.

Operation Notes and Details

Transit agency supervisors should train and educate their lift attendants on the proper use and operation of the wheelchair lift if it is not possible for the attendants to review the safety precautions and operation procedures with the wheelchair lift sales representative.

The lift operator's manual must be stored in the lift-mounted manual storage pouch at all times.

Preventive Maintenance:

Maintenance is necessary to ensure safe and trouble free lift operation. General preventive lift maintenance consisting of careful inspections of the lift system and cleaning the lift should

be a part of your transit agency's daily lift service program. Simple inspections can detect potential lift operational problems.

Regular preventive maintenance will reduce potential lift operation downtime and increase the service life of the lift, as well as possibly detecting potential hazards.

Exposure to harsh weather elements, environmental conditions, or heavy usage may require more frequent maintenance and lubrication procedures.

Preventive maintenance visual inspections do not take the place of the procedures specified in the Maintenance and Lubrication Schedule provided on pages 39-

47 of this manual. Refer to the Maintenance and Lubrication section for further details.

WARNING

Maintenance and lubrication procedures must be performed by authorized service personnel as specified in this manual. Failure to do so may result in serious bodily injury and/or property damage.

Lift Operating Instructions

WARNING

Read and become familiar with all lift operation safety precautions, operation notes and details, operating instructions and manual operating instructions prior to operating the lift. If this information is not fully understood, contact The Braun Corporation immediately. Failure to do so may result in serious bodily injury and/or property damage.

WARNING

Whenever a passenger is on the platform, the:

- Passenger must be positioned fully inside yellow boundaries
- Wheelchair brakes must be locked
- Inner roll stop and outer barrier must be UP

Failure to follow these rules may result in serious bodily injury and/or property damage.

Before lift operation, park the vehicle on a level surface, away from vehicular traffic. Place the vehicle transmission in "Park" and engage the parking brake. Vehicle engine must be running.

Lift Operating Instructions photos depict lift model NL9171B. Instructions and procedures are identical for all NL-2 lift models. Lift-posted Warnings and Lift Operating Instructions decal 30695 provides lift operating instructions. Replace any missing, worn or illegible decals.

If your lift does not function as intended or a visual or audible warning signal is activated, review the NHTSA Operations Checklist on page 32. Refer to the Manual Operating Instructions on pages 33-35 to manually operate lift.

Lift Operating Instructions



Open Door(s) and Secure

To Unfold Platform:

Stand clear and press the UNFOLD switch until the platform stops (reaches floor level - unfolds fully). Release switch.

Note: In event platform does not unfold, press FOLD switch to release Lift-Tite™ latches.



To Unload Passenger:

1. Read Note below! Load passenger onto platform and lock wheelchair brakes.

Note: Passenger must be positioned fully inside yellow boundaries, outer barrier must be UP and outer barrier latch must be engaged.



Lift Operating Instructions



To Unload Passenger (continued):

2. Press DOWN switch until the entire platform reaches ground level (see Photo B) and the outer barrier unfolds fully (ramp position). See Photo C. Release switch.




3. Unlock wheelchair brakes and unload passenger from platform.

Note: Outer barrier must be fully unfolded (ramp position) until the entire wheelchair (or standee) has crossed the outer barrier. See Photos E and F on page 30 also.




Lift Operating Instructions

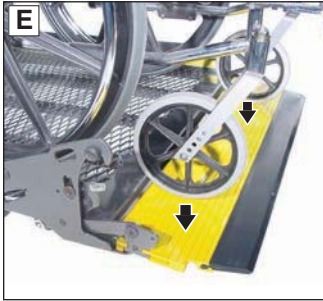


To Load Passenger:

1. Read Notes below! Load passenger onto platform and lock wheelchair brakes. See Photo G.

Note: Outer barrier must be fully unfolded (ramp position) until the entire wheelchair (or standee) has crossed the outer barrier. See Photos E and F.

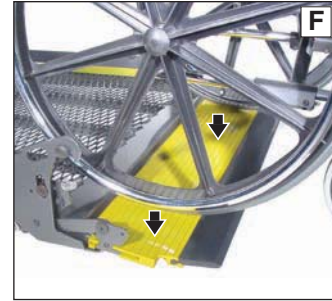

Note: Passenger must be positioned fully inside yellow boundaries.

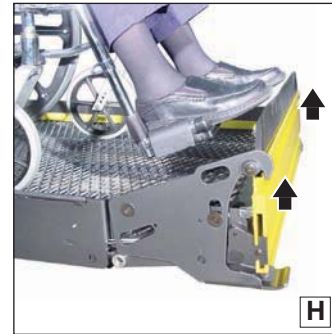

Lift Operating Instructions



To Load Passenger (continued):

2. Press UP switch (Photo I) to fold outer barrier UP fully (vertical - see Photo H), and raise the platform to floor level. See Photo J. Release switch.
3. Unlock wheelchair brakes and unload passenger from platform.



To Fold Platform:

Press FOLD (In) switch until platform stops (fully folded). See Photos K and L. Release switch.



Close Door(s)

NHTSA Operations Checklist

The following operations have been verified upon installation. This operational checklist can be used at any time to verify the lift is fully functional.

Verified:

- ☐ Vehicle movement is prevented unless the lift door is closed, ensuring the lift is stowed.
- ☐ Lift operation shall be prevented unless the vehicle is stopped and vehicle movement is prevented.
- ☐ The platform will not fold/stow if occupied.
- ☐ The inner roll stop will not raise if occupied.
- ☐ The outer barrier will not raise if occupied.
- ☐ Verify platform lighting* when lift is deployed and pendant illumination when lift is powered.
- ☐ A visual and audible warning will activate if the threshold area is occupied when the platform is at least one inch below floor level.
- ☐ Platform movement is prohibited beyond the position where the inner roll stop is fully deployed (up).
- ☐ Platform movement shall be interrupted unless the outer barrier is deployed (up).

WARNING

Discontinue lift use immediately if any lift or vehicle interlock does not operate properly. Failure to do so may result in serious bodily injury and/or property damage.


*Public use vehicle manufacturers are responsible for complying with the lift lighting requirements in Federal Motor Vehicle Safety Standard No. 404, Platform Lift Installations in Motor Vehicles (49 CFR 571.404).

Manual Operating Instructions


Instructions are provided for all steps that differ from standard lift operation procedures. Manual Operation decal 32940 (posted on pump cover)

provides manual operating instructions also. Note: Inner roll stop and outer barrier automatically function during manual operation.

Refer to the Lift Operating Instructions for all normal lift operation procedures. Follow all Lift Operation Safety Precautions!

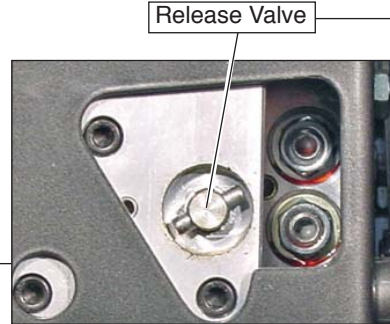

Note: Close backup pump release valve securely before operating electric pump.

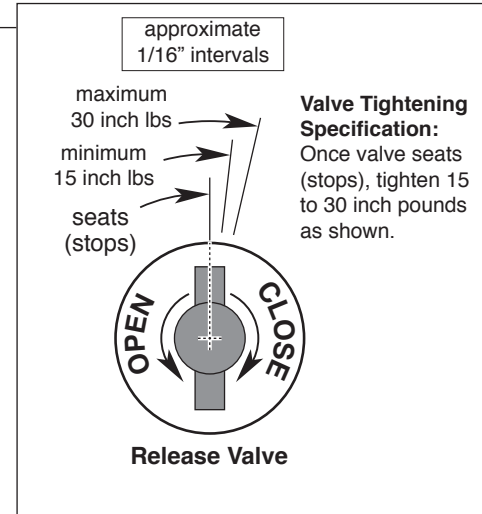

Manual Operating Instructions

Refer to release valve photos and illustration on page 33.



To Unfold Platform (Out):

Using hand pump handle (Photo B):

1. Close hand pump valve (place slotted end of pump handle onto backup pump release valve and turn clockwise). See Photo C.
2. Insert handle in pump and stroke until platform folds fully (stops). See Photo E on page 35.
3. Open hand pump valve (turn counterclockwise) until platform reaches floor level. Open 1/2 turn only.
4. Close hand pump valve (turn clockwise).

Note: Valve must be tight, but do not overtighten.


Down (To Lower):

Place slotted end of pump handle onto backup pump release valve and turn counterclockwise (open 1/2 turn only) until the platform reaches ground level and outer barrier unfolds.



Manual Operating Instructions

Refer to release valve photos and illustration on page 33.

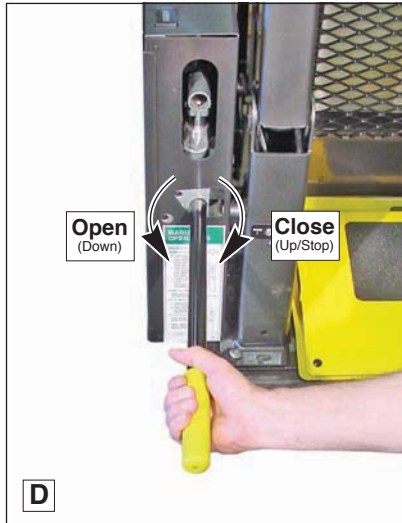

Up (To Raise):

Using hand pump handle:

- Place slotted end of pump handle onto back-up pump release valve and turn clockwise to close securely. See Photo D.


Note: Valve must be tight, but do not overtighten.

- Insert handle into back-up pump and stroke until platform reaches floor level (see Photo E).

To Fold Platform (In):

Insert handle into back-up pump and stroke until platform stops (folds fully). See Photo E.

Note: Close back-up pump release valve securely before operating electric pump. Store pump handle in clamps shown in Photo B.

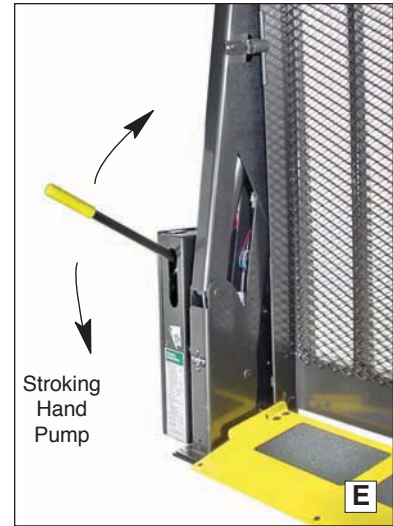

Decals and Antiskid

⚠ WARNING

Replace missing, worn or illegible decals.
Failure to do so may result in serious bodily injury and/or property damage.

Note: Clean surfaces as detailed on page 38 before posting decals.


25675

⚠ WARNING

Lift installation and servicing prohibited by anyone who has not been certified by The Braun Corporation Sales and Service School. Certified service technicians should call 1-800-THE LIFT to receive applicable installation/service manual. Failure to follow this policy may result in serious bodily injury and/or property damage.

25675

21494

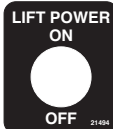

Decals

The lift is only as safe as the operator. Replace any missing, worn or illegible decals! Part numbers are provided for decals. Inspect your lift for missing, worn or illegible decals. Call 1-800-THE LIFT® for replacements.

31045

OPERATING INSTRUCTIONS

Read warnings and operate lift as outlined on LIFT OPERATING INSTRUCTIONS decal. Lift operating instructions apply to wheelchair passengers and standees.

Standees: Standees must stand at center of platform (fully inside yellow boundaries), grip handrails and lower head.

Interlocks: Interlocks vary in type and operation. Interlocks must be operational as defined in NHTSA Operations Checklist.

DOT - Public Use Lift

31045

32940

MANUAL OPERATION

Using hand pump handle:

TO UNFOLD PLATFORM:

1. Close hand pump valve (turn clockwise).
2. Insert handle in pump and stroke until platform folds fully (stops).
3. Open hand pump valve (turn counterclockwise) until platform reaches floor level. Open 1/2 turn only.
4. Close hand pump valve (turn clockwise).

DOWN (TO LOWER):

Open hand pump valve (turn counterclockwise). Open 1/2 turn only.

UP (TO RAISE):

1. Close hand pump valve (turn clockwise).
2. Insert handle in pump and stroke platform reaches floor level.

TO FOLD PLATFORM:

Insert handle in pump and stroke.

Note: Close valve before operating electric pump.

DOT - Public Use Lift

32940

33221



30695

⚠ WARNING

- Read manual before operating lift.
 - Load and unload on level surface only.
 - Engage vehicle parking brake before operating lift.
 - Provide adequate clearance outside of vehicle to:
 - Do not open lift if you suspect lift damage, wear or the abnormal operation.
 - Keep operator and bystanders clear of area in which lift operates.
- Whenever a wheelchair passenger is on the platform, the:
- Passenger must be positioned fully inside yellow boundaries.
 - Wheelchair brakes must be locked.
 - Inter roll stop and outer barrier must be up.
- Failure to follow these rules may result in serious bodily injury and/or property damage.

LIFT OPERATING INSTRUCTIONS

OPEN DOOR(S) AND SECURE

TO UNFOLD PLATFORM:

Stand clear and press UNFOLD switch until platform stops (reaches floor level).

When the wheel platform does not unfold, press FOLD button to release Lift Trip™ latch.

TO UNLOAD PASSENGER:

1. Load passenger onto platform and lock wheelchair brakes.
2. Press DOWN switch until platform reaches ground level and outer barrier must be up.
3. Unlock wheelchair brakes and unload passenger from platform.

TO LOAD PASSENGER:

1. Load passenger onto platform and lock wheelchair brakes.
2. Press UP switch to fold outer barrier up and raise platform to floor level.
3. Unlock wheelchair brakes and unload passenger from platform.

TO FOLD PLATFORM:

Press FOLD switch until platform stops. Release switch.

CLOSE DOOR(S)

DOT - Public Use Lift

30695

Decals and Antiskid

Decals

31130



31131



32942



32943



34267



18229




31132




32094



32201

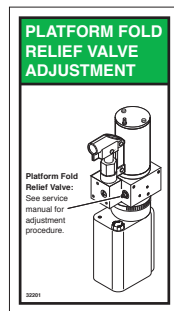

22249

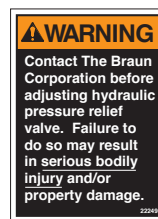

25652



30236R: 1" Wide Yellow Boundary Tape (order by the inch as needed)



Decals and Antiskid

WARNING

Replace missing or worn antiskid. Failure to do so may result in serious bodily injury and/or property damage.

Inspect your lift for any missing or worn antiskid. Order as needed.

Note: Clean surfaces with isopropyl alcohol before decal or antiskid application. Use a clean cloth or paper towels. Do not use oily shop rags. Wipe surface free of residue with dry portion of cleaning cloth.

Antiskid

Available Antiskid		
Size	Color	Part No.
2" x 12"	Black	#24172-BK
2" x 12"	Yellow	#24172-YL
3" x 8"	Black	#31188-BK
3" x 12"	Black	#24173-BK
3" x 12"	Yellow	#24173-YL
6" x 12"	Black	#24174-BK
6" x 12"	Yellow	#24174-YL

Maintenance and Lubrication

Proper maintenance is necessary to ensure safe, trouble free operation. Inspecting the lift for any wear, damage or other abnormal conditions should be a part of the transit agency daily service program. Simple inspections can detect potential problems.

The maintenance and lubrication procedures specified in the following schedule must be performed by a Braun authorized service representative at the scheduled intervals according to the number of cycles.

NL-2 Series lifts are equipped with hardened pins and self-lubricating bushings to decrease wear, provide smooth operation and extend the service life of the lift.

When servicing the lift at the recommended intervals, inspection and lubrication procedures specified in the previous sections should be repeated. Clean components and the surrounding area before applying lubricants. LPS2 General

Purpose Penetrating Oil is recommended where Light Oil is called out. Use of improper lubricants can attract dirt or other contaminants which could result in wear or damage to the components. Platform components exposed to contaminants when lowered to the ground may require extra attention.

Lift components requiring grease are lubricated during assembly procedures. When these components are replaced, grease must be applied during installation procedures. Specified lubricants are available from The Braun Corporation (part numbers provided on page 40).

WARNING

Maintenance and lubrication procedures must be performed as specified by an authorized service technician. Failure to do so may result in serious bodily injury and/or property damage.

Maintenance and Lubrication

All listed inspection, lubrication and maintenance procedures should be repeated at 750 cycle intervals following the scheduled 4500 cycle maintenance procedures. These intervals are a general guideline for scheduling maintenance procedures and will vary according to lift use and conditions. Lifts exposed to severe conditions (weather, environment, contamination, heavy usage, etc.) may require inspection and maintenance procedures to be performed more often than specified.


Cycle Counter: NL-2 Series lift models are equipped with a cycle counter located on the top

of the pump module. This cycle counter allows the lift attendant/operator to easily track the number of cycles during daily inspections of the lift.

Discontinue lift use immediately if maintenance and lubrication procedures are not properly performed, or if there is any sign of wear, damage or improper operation. Contact your sales representative or call The Braun Corporation at 1-800-THE LIFT®. One of our national Product Support representatives will direct you to an authorized service technician who will inspect your lift.

See the Maintenance/Lubrication Schedule for recommended applications per number of cycles.

Lubricant	Type	Specified (recommended) Lubricant	Available Amount	Braun Part No.
LO - Light Oil	Light Penetrating Oil (30 weight or equivalent)	LPS2, General Purpose Penetrating Oil	16 oz. Aerosol Can	15807
DE - Door-Ease	Stainless Stick Style (tube)	Door-Ease Stick (tube)	1.68 oz.	15806
LG - Light Grease	Light Grease (Multipurpose)	Lubriplate	14 oz. Can	15805

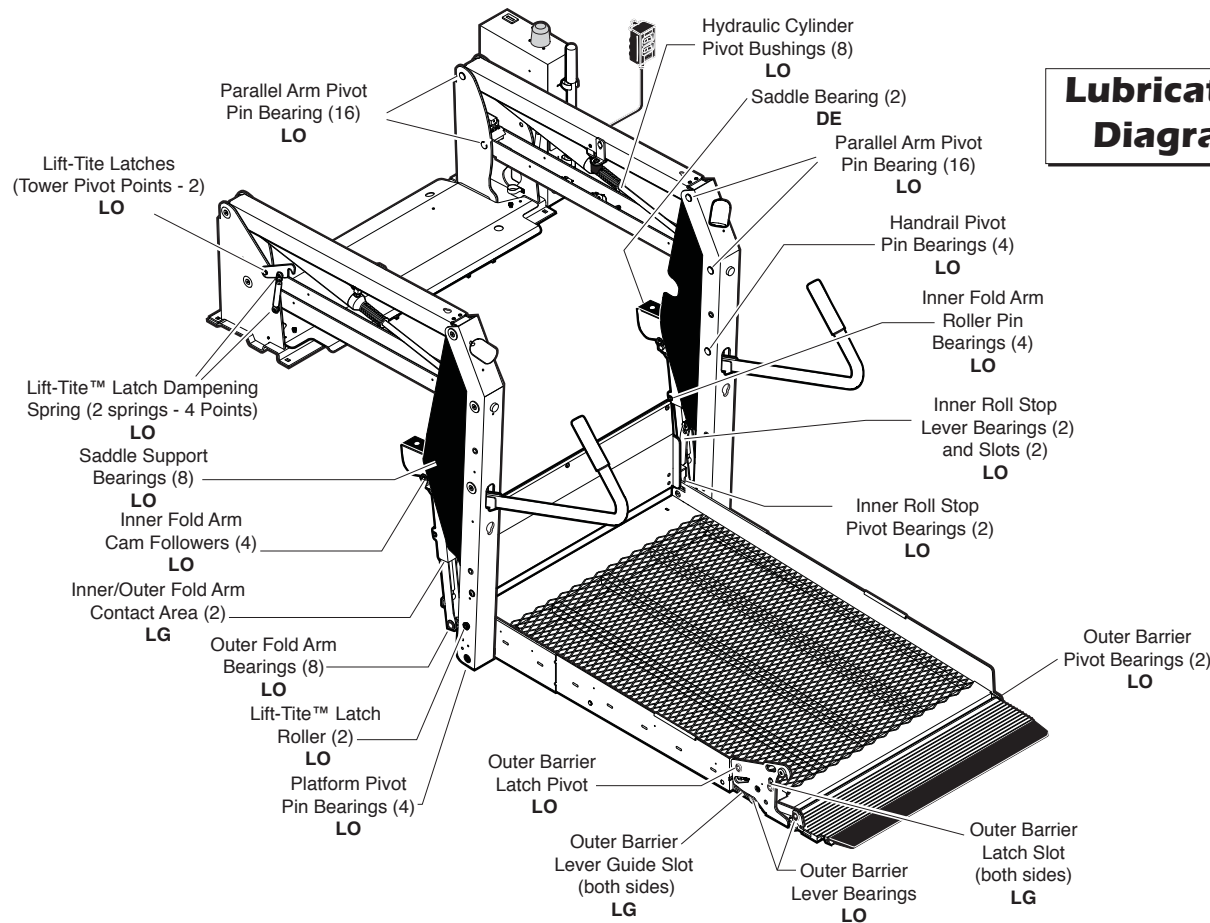

Lubrication Diagram

Maintenance and Lubrication Schedule

750 Cycles	Outer barrier pivot points (2)	Apply Light Oil - See Lubrication Diagram
	Outer barrier latch pivot point	Apply Light Oil - See Lubrication Diagram
	Outer barrier latch slot	Apply Light Grease to both sides of slot. See Lubrication Diagram
	Outer barrier lever bearings (2)	Apply Light Oil - See Lubrication Diagram
	Lift-Tite™ latches (tower pivot points - 2)	Apply Light Oil - See Lubrication Diagram
	Lift-Tite™ latch gas (dampening) spring pivot points (2 springs - 4 points)	Apply Light Oil - See Lubrication Diagram
	Inspect Lift-Tite™ latches and gas springs for wear or damage (bent, deformed or misaligned), positive securement (external snap rings) and proper operation	Resecure, replace damaged parts or otherwise correct as needed. Note: Apply Light Grease to Lift-Tite™ latch tower pivot point if replacing latch.
	Inspect outer barrier for proper operation	Correct or replace damaged parts.
	Inspect outer barrier latch for proper operation, positive securement, and detached or missing spring	Correct or replace damaged parts and/or relubricate. See Lubrication Diagram
continued	Adjust fold pressure and outer barrier fold pressure (if applicable)	See applicable service manual

Maintenance and Lubrication Schedule

continued 750 Cycles	Verify NHTSA Operations Checklist	See NHTSA Operations Checklist
	Inspect lift for wear, damage, or any abnormal condition	Correct as needed.
	Inspect lift for rattles	Correct as needed
1500 Cycles continued	Perform all procedures listed in previous section also	
	Inner/outer fold arms (2)	Apply grease (synthetic) to contact areas between inner/outer fold arms. See Lubrication Diagram.
	Platform pivot pin bearings (4)	Apply Light Oil - See Lubrication Diagram
	Outer fold arm bearings (8)	Apply Light Oil - See Lubrication Diagram
	Inner roll stop pivot bearings (2)	Apply Light Oil - See Lubrication Diagram
	Inner roll stop lever bearings (2)	Apply Light Oil - See Lubrication Diagram
	Inner roll stop lever slot (2)	Apply Light Oil - See Lubrication Diagram
	Saddle support bearings (8)	Apply Light Oil - See Lubrication Diagram
	Inner fold arm roller pin bearings (4)	Apply Light Oil - See Lubrication Diagram

Maintenance and Lubrication Schedule

continued

**1500
Cycles**

continued

Inner fold arm cam followers (4)	Apply Light Oil - See Lubrication Diagram
Parallel arm pivot pin bearings (16)	Apply Light Oil - See Lubrication Diagram
Handrail pivot pin bearings (4)	Apply Light Oil - See Lubrication Diagram
Hydraulic cylinder pivot bushings (8)	Apply Light Oil - See Lubrication Diagram
Outer barrier lever guide slot	Apply Light Grease to both sides of slot. See Lubrication Diagram.
Inspect Lift-Tite™ latch rollers for wear or damage, positive securement and proper operation (2)	Correct, replace damaged parts and/or relubricate.
Inspect inner roll stop for: • Wear or damage • Proper operation. Roll stop should just rest on top surface of the threshold plate. • Positive securement (both ends)	Resecure, replace or correct as needed. See Platform Angle Instructions and Tower Micro-switch Adjustment Instructions.
Inspect handrail components for wear or damage, and for proper operation	Replace damaged parts
Inspect microswitches for securement and proper adjustment.	Resecure, replace or adjust as needed. See Microswitch Adjustment Instructions.

Maintenance and Lubrication Schedule

continued	Make sure lift operates smoothly	Realign towers and vertical arms. Lubricate or correct as needed.
1500 Cycles	Inspect external snap rings: • Outer fold arm (6) • Lift-Tite™ latch roller (2) • Lift-Tite™ latch gas (dampening) spring (4) • Inner fold arm cam followers (4) • Inner fold arm roller pins (4) • Outer barrier hydraulic cylinder mounting pin (2) • Inner roll stop lever bracket pins (2)	Resecure or replace if needed.
	Inspect inner roll stop locks (2) and torsion springs (2) for wear or damage and for proper operation.	Replace damaged parts and resecure as needed. Apply Light Oil to inner roll stop lock pivot points.
	Inspect outer fold arm pins (2), axles (2) and bearings (8) for wear or damage and positive securement	Replace damaged parts and resecure as needed. Apply Light Oil.
	Remove pump module cover and inspect: • Hydraulic hoses, fittings and connections for wear or leaks • Harness cables, wires, terminals and connections for securement or damage • Relays, fuses, power switch and lights for securement or damage	Resecure, replace or correct as needed.

Maintenance and Lubrication Schedule

4500 Cycles	Perform all procedures listed in previous section also	
	Inspect cotter pins on platform pivot pin (2)	Resecure, replace or correct as needed
	Hydraulic Fluid (Pump) - Check level. Note: Fluid should be changed if there is visible contamination. Inspect the hydraulic system (cylinder, hoses, fittings, seals, etc.) for leaks if fluid level is low.	Use Braun 32840-QT hydraulic fluid (Exxon® Uni-vis HVI 26). Do not mix with Dextron III or other hydraulic fluids. Check fluid level with platform lowered fully. Fill to maximum fluid level indicated on reservoir (specified on decal). Do not overfill. If fluid level decal is not present - measure 1-3/8" from the fill port to locate fluid level.
	Inspect cylinders, fittings and hydraulic connections for wear, damage or leaks	Tighten, repair or replace if needed.
	Inspect outer barrier cylinder hose assembly (hose, fasteners, connections, etc.) for wear, damage or leakage	Tighten, repair or replace if needed.
	Inspect parallel arms, bearings and pivot pins for visible wear or damage	Replace if needed.
continued	Inspect parallel arm pivot pin mounting bolts (8)	Tighten or replace if needed.
	Inspect platform pivot pins, bearings and vertical arms for wear, damage and positive securement	Replace damaged parts and resecure as needed. Apply Light Grease during reassembly procedures.

Maintenance and Lubrication Schedule

continued	Inspect inner/outer fold arms, saddle, saddle support and associated pivot pins and bearings for visible wear or damage Replace if needed.
4500 Cycles	Inspect gas springs (cylinders) for wear or damage, proper operation and positive securement Tighten, replace or correct as needed
	Inspect saddle bearing (UHMW - 2) Apply Door-Ease or replace if needed. See Lubrication Diagram.
	Inspect vertical arm plastic covers Resecure or replace if needed.
	Inspect power cable Resecure, repair or replace if needed.
	Mounting Check to see that the lift is securely anchored to the vehicle and there are no loose bolts, broken welds, or stress fractures.
	Decals and Antiskid Replace decals if worn, missing or illegible. Replace antiskid if worn or missing. See Decals and Antiskid section on pages 38-40.
Consecutive 750 Cycle Intervals	Repeat all previously listed inspection, lubrication and maintenance procedures at 750 cycle intervals.

Limited Warranty

WARRANTY COVERAGE AND WARRANTY COVERAGE TIME PERIODS

The Braun Corporation (“Braun”) warranty covers certain parts of this wheelchair lift for three (3) years or 10,000 cycles and the cost of labor to repair or replace those parts for one (1) year or 3,000 cycles. If The Braun Corporation receives the warranty registration card within 20 days after the lift is put into service, the warranty labor coverage will increase from one (1) year or 3,000 cycles to three (3) years or 10,000 cycles. In addition, providing the warranty registration card is returned as noted above, the following lift’s power train parts are warrantied for five (5) years or 15,000 cycles: Cable, Cylinder, Flow Control, Gear Box, Motor, Pump, Hydraulic Hose and Fittings. This limited warranty covers substantial defects in materials and workmanship of the lift, provided that the lift is operated and maintained properly and in conformity with the owner’s manual. The warranty period begins on the date that the product is delivered to the first retail purchaser by an independent, authorized dealer of Braun, or, if the dealer places the product into any type of service prior to retail sale, on the date the dealer first places the product in such service. This limited warranty applies only to the first purchaser. It may not be transferred.

WHAT BRAUN WILL DO TO CORRECT PROBLEMS

In the event that a substantial defect in material or workmanship, attributable to Braun, is found to exist during the first year of warranty coverage, it will be repaired or replaced, at Braun’s option, without charge for parts or labor to the owner, in accordance with the terms, conditions and limitations of this limited warranty. If the substantial defect in material or workmanship, attributable to Braun, is found to exist during the second or third year of warranty coverage, it will be repaired or replaced, at Braun’s option, without charge to the owner for parts, only, in accordance with the terms, conditions and limitations of this limited warranty.

Limited Warranty

Providing the warranty card is returned within 20 days as outlined above, the labor warranty period will be extended by two years of coverage in accordance with the terms, conditions, and limitations of this limited warranty. In addition, if a substantial defect in material or workmanship, attributable to Braun, is found to exist during the fourth or fifth year of warranty coverage to the following lift's power train parts: Cable, Cylinder, Flow Control, Gear Box, Motor, Pump, Hydraulic Hose and Fittings, it will be repaired or replaced, at Braun's option, without charge to the owner for parts, only, in accordance with the terms, conditions and limitations of this limited warranty. The cost of labor for repair or replacement at any time after the warranty coverage detailed above is the sole responsibility of the owner.

Braun's obligation to repair or replace defective materials or workmanship is the sole obligation of Braun under this limited warranty. Braun reserves the right to use new or remanufactured parts of similar quality to complete any work, and to make parts and design changes from time to time without notice to anyone. Braun reserves the right to make changes in the design or material of its products without incurring any obligation to incorporate such changes in any previously manufactured product. Braun makes no warranty as to the future performance of this product, and this limited warranty is not intended to extend to the future performance of the product. In addition, the owner's obligation to notify Braun, or one of its authorized, independent dealers, of a claimed defect does not modify any obligation placed on the owner to contact Braun directly when attempting to pursue remedies under state or federal law.

Limited Warranty

LIMITATIONS, EXCLUSIONS AND DISCLAIMER OF IMPLIED WARRANTIES

ANY IMPLIED WARRANTY THAT IS FOUND TO ARISE BY WAY OF STATE OR FEDERAL LAW, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS, IS LIMITED IN DURATION TO THE TERMS OF THIS LIMITED WARRANTY AND IS LIMITED IN SCOPE OF COVERAGE TO THE SCOPE OF COVERAGE OF THIS LIMITED WARRANTY. Braun disclaims any express or implied warranty, including any implied warranty of fitness or merchantability, on items excluded from coverage as set forth in this limited warranty. Braun makes no warranty of any nature beyond that contained in this limited warranty. No one has authority to enlarge, amend or modify this limited warranty, and Braun does not authorize anyone to create any other obligation for it regarding this product. Braun is not responsible for any representation, promise or warranty made by any independent dealer or other person beyond what is expressly stated in this limited warranty. Any selling or servicing dealer is not Braun's agent, but an independent entity.

BRAUN SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES THAT MAY RESULT FROM BREACH OF THIS LIMITED WARRANTY OR ANY IMPLIED WARRANTY. THIS EXCLUSION OF CONSEQUENTIAL AND INCIDENTAL DAMAGES SHALL BE INDEPENDENT OF ANY FAILURE OF THE ESSENTIAL PURPOSE OF ANY WARRANTY, AND THIS EXCLUSION SHALL SURVIVE ANY DETERMINATION THAT THIS LIMITED WARRANTY OR ANY IMPLIED WARRANTY HAS FAILED OF ITS ESSENTIAL PURPOSE. This warranty does not cover, and in no event shall Braun be liable for towing charges, travel, lodging, or any other expense incurred due to the loss of use of the product or other reason.

Limited Warranty

Some states do not allow limitations on how long an implied warranty lasts, or the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you.

HOW TO GET SERVICE

To obtain warranty service the owner must do all of the following:

1. Notify an authorized service center, of the claimed defect attributable to Braun, within the warranty coverage period designated above
2. Provide the notification mentioned in (1), above, within ten (10) days of when the owner discovered, or should have discovered, the claimed defect
3. Promptly schedule an appointment with and take the product to an authorized service center for service.
4. Pay any transportation costs and all expenses associated with obtaining warranty service.

Since Braun does not control the scheduling of service work at the independent dealerships you may encounter some delay in scheduling or completion of work. If you need assistance you may contact Braun, at 631 West 11th Street, Winamac, Indiana 46996; 1-800-THE-LIFT, (843-5438).

If two (2) or more service attempts have been made to correct any covered defect that you believe impairs the value, use or safety of the product, or if it has taken longer than thirty (30) days for repairs to be completed, you must, to the extent permitted by law, notify Braun directly, in writing, at the above address, of the unsuccessful repair(s) of the alleged defect(s) so that Braun can become directly involved in providing service pursuant to the terms of this limited warranty.

Limited Warranty

WHAT IS NOT COVERED

This Limited Warranty does not cover any of the following: defects in materials, components or parts of the product not attributable to Braun, any material, component or part of the product that is warranted by another entity (Note: the written warranty provided by the manufacturer of the material, component or part is the direct responsibility of that manufacturer); items that are added or changed after the product leaves Braun's possession; additional items installed at any dealership, or other place of business, or by any other party, other than Braun; normal wear, tear, usage, maintenance, service, periodic adjustments, the effects of condensation or moisture from condensation; mold or any damage caused by mold; imperfections that do not affect the product for its intended purpose; items that are working as designed but that you are unhappy with; problems related to mis-operation, misuse, mishandling, neglect or abuse, including failure to maintain the product in accordance with the owner's manual, or other routine maintenance such as inspections, lubricating, adjustments, tightening of screws, sealing, wheel alignments or rotating tires; damage due to accident or collision, including any acts of weather or damage or corrosion due to the environment; theft, vandalism, fire, or other intervening acts not attributable to Braun; damage resulting from tire wear or tire failure; defacing, scratches, dents or chips on any interior or exterior surface of the product, including those caused by rocks or other road hazards, damage caused by off road use, overloading or alteration of the product, or any of its components or parts;

Defects and/or damage to interior and exterior surfaces and other appearance items may occur at the factory or when the product is in transit. These items are usually detected and corrected at the factory or by a dealer prior to delivery to the purchaser. You must inspect the product for this type of damage when you take delivery. If you find any such defect or damage you must notify the selling dealer, or Braun, at the time

Limited Warranty

of delivery to have these items covered by this limited warranty and to have work performed on the items at no cost to you as provided by this limited warranty.

EVENTS DISCHARGING BRAUN FROM OBLIGATION UNDER WARRANTY

The following shall completely discharge Braun from any express or implied warranty obligation to repair or replace anything and void this warranty: misuse, neglect, collision, accidents, failure to provide routine maintenance (See Owner's Manual), unauthorized alteration, off road use, Acts of Nature, damage from weather or the environment, theft, vandalism, tampering, fire, explosions, overloading the product and odometer tampering.

LEGAL REMEDIES

Any action to enforce any portion of this limited warranty, or any implied warranty, must be commenced within six (6) months after expiration of the warranty coverage period designated above or the action will be barred because of the passage of time. Any performance of repairs shall not suspend this limitation period from expiring. Any performance of repairs after the warranty coverage period has expired, or performance of repairs regarding any thing excluded from coverage under this limited warranty shall be considered "good will" repairs, and they will not alter the terms of this limited warranty, or extend the warranty coverage period or the filing limitation period in this paragraph. In addition, since it is reasonable to expect that the product will need some service during the warranty period; this warranty does not extend to future performance. It only sets forth what Braun will do and does not guarantee anything about the product for any time period.

Limited Warranty

Nothing in this warranty, or any action of Braun, or any agent of Braun, shall be interpreted as an extension of any warranty period or the filing limitation period in this paragraph. Some states do not allow a reduction in the statute of limitations, so this reduction may not apply to you.

WARRANTY REGISTRATION and MISCELLANEOUS

Your warranty registration records should be completed and delivered to the appropriate companies, including the Braun Delivery Checklist & Warranty form. That form must be returned to Braun within twenty (20) days of purchase. The Braun warranty will not be registered unless this warranty registration is completed and received by Braun. Failure to file this warranty registration with Braun will not affect your rights under this limited warranty as long as you can present proof of purchase, but it can cause delays in obtaining the benefits of this limited warranty, and it changes the start date of the warranty to the date of final assembly of the product by Braun.

Braun agrees to repair or replace any of its factory installed parts found to have substantial defects within the appropriate warranty period designated above, provided that the repair is authorized by Braun and carried out by an authorized service center (a Braun labor schedule determines the cost allowance for repairs). Braun will not honor any warranty claim for repairs or replacement of parts unless the claim is submitted with the appropriate paperwork, and the work is completed by an independent, factory authorized service center. The appropriate paperwork can be obtained by written or phone contact with Braun at the contact information in this warranty.

Limited Warranty

Braun reserves the right to designate where any warranty work can be performed. Braun also reserves the right to examine any defective workmanship or part prior to giving any authorization for warranty work. Braun's return authorization procedure must be adhered to in order to process any warranty claims.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS. YOU MAY ALSO HAVE OTHER RIGHTS THAT VARY FROM STATE TO STATE.

"Providing Access to the World"®



Operator's Manual for:

NL **Millennium 2**

Public Use Wheelchair Lifts

Patent #5,261,779

Patent #6,599,079

Patent #7,422,408

Patent #6,065,924

Patent #6,692,217

Patent #7,444,995

Patent #6,238,169

Patent #6,739,824

Patent #7,530,226

Patent #6,464,447

Patent #7,306,422

BRAUN
MILLENNIUM 2
SERIES

Braun Millennium 2 Series

All illustrations, descriptions and specifications in this manual are based on the latest product information available at the time of publication. The Braun Corporation reserves the right to make changes at any time without notice.

33703 Rev. G March 2010

© The Braun Corporation